

**DRI® Benzodiazepine Cross-Reactivity Tables**  
**For catalog #s 0039 (starting with lot # 58194699) and 0040 (starting with lot # 58194700)**

**POSITIVE COMPOUNDS**

The following compounds tested POSITIVE on the DRI® DAU Benzodiazepine assay at the 200 ng/mL cutoff.

Positive Compounds	Trade Name	Concentration Tested (ng/mL)
Alprazolam	Tafil, Valeans, Xanax, Xandor	105
7-Aminoclonazepam		2500
7-AminoFlunitrazepam		3100
Bromazepam	Lectopam	225
Chlordiazepoxide	Libritabs, Librium, Sereen	1100
Clobazam		145
Clonazepam	Klonopin	500
Clorazepate	Gen-XENE, Tranxene	120
Delorazepam		110
Desmethyldiazepam (nor-diazepam)	Nordaz, Calmday	100
Diazepam	Valium, Valrelease, Vazepam	95
Flunitrazepam	Rohypnol	175
Flurazepam	Dalmane, Durapam	140
Lorazepam	Ativan	1000
Lormetazepam		225
Medazepam	Nobrium	225
Midazolam		900
N-DesmethylFlunitrazepam		3100
Nitrazepam		175
Norfludiazepam		115
Prazepam	Centrax	110
Temazepam	Restoril, Razepam, Temaz	125
Triazolam	Halcion	125

## NEGATIVE COMPOUNDS

The following compounds tested NEGATIVE on the DRI® DAU Benzodiazepine assay at the 200 ng/mL cutoff.

Negative Compounds	Trade Name	Concentration Tested (µg/mL)
Acetaminophen	Anacin, Datri Extra, Liquiprin, Panadol, Temptra, Tylenol	1,000
Acetylsalicylic acid	Aspirin, Bufferin	1,000
Amphetamine		1,000
Caffeine	Coffe-Break, Durivitan, No-Doz, Percoffedrinol N, Pro-Plus, Vivarin	100
Codeine		1,000
Dextromethorphan	Benylin DM, Delsym, Hold, Perfussin 8 hr, Mediquell, Sucrets	1,000
Diphenhydramine	Banophen, Benadryl, Diphedryl, Simply Sleep	500
Fluoxetine		500
Gabapentin	Neurontin	1,000
Methadone	Dolophine	1,000
Morphine		200
Nor-Fluoxetine		500
Nor-Sertraline		1000
Oxaprozin	Daypro	50
Paroxetine		500
Phencyclidine		1,000
Propoxyphene	Darvon	1,000
Secobarbital	Seconal	1,000
Sertraline		500