

CEDIA® Heroin Metabolite (6-Acetylmorphine) Cross-Reactivity Tables
For catalog #s 100107 and 100108

POSITIVE COMPOUNDS

No compounds to date have been tested, other than 6-Acetylmorphine, that produce positive results in the CEDIA DAU 6-AM assay at the 10 ng/mL cutoff.

Positive Compounds	Trade Name	Concentration Tested (ng/mL)

NEGATIVE COMPOUNDS

The following compounds tested NEGATIVE on the CEDIA® DAU 6-AM assay at the 10 ng/mL cutoff.

Negative Compounds	Trade Name	Concentration Tested (ng/mL)
10,11-Dihydrocarbamazepine		85,000
11-nor- Δ^9 -THC-COOH		10,000
Acetaminophen (paracetamol)	Anacin, Datri Extra, Liquiprin, Panadol, Tempra, Tylenol	500,000
Acetylsalicylic acid	Aspirin	500,000
Amitriptyline	Elavil, Endep, Enovil, Emitrip	500,000
Amoxicillin	Amoxil, Polymor, Trimox, Wymox	500,000
Amphetamine		1,000,000
Benzotropine Mesylate	Cogentin	500,000
Benzoylcegonine		100,000
Brompheniramine	Dimetane, Dimetapp, Nasahist, ND-Stat, Oraminic II	75,000
Caffeine	Coffe-Break, Durivitan, No-Doz, Percoffedrinol N, Pro-Plus, Vivarin	500,000
Captopril	Capoten	500,000
Chlordiazepoxide	Libritabs, Librium, Sereen	100,000
Chlorpromazine	Thorazine	10,000
Cimetidine	Tagamet	500,000
Codeine		500,000
Codeine-6-Glucuronide		500,000
Desipramine	Norpramin, Petrofrane	500,000

Negative Compounds	Trade Name	Concentration Tested (ng/mL)
Dextromethorphan	Benlyn DM, Delsym, Hold, Perfussin 8 hr, Mediquell, Sucrets	100,000
Diazepam	Valium, Valrelease, Vazepam	100,000
Digoxin	Lanoxin	100,000
Dihydrocodeine	DHC Plus, Synalgos-DC	500,000
Diphenhydramine	Banophen, Benadryl, Diphedryl, Simply Sleep	50,000
Doxepine HCL	Adapin, Sinequan	100,000
Enalapril	Vasotec	500,000
Fluoxetine	Prozac, Sarafem	500,000
Haloperidol	Haldol, Peridol	100,000
Heroin HCl		80
Hydrocodone	Lortab, Vicodin	300,000
Hydromorphone	Dilaudid	10,000
Hydroxyzine 2HCl	Anxanil, Apo-Hydroxyzine, Atarax, Hyzine-50	500,000
Ibuprofen	Advil, Haltran, Mediprene, Motrin, Nuprin, Proen, Rufen	500,000
Imipramine	Tofranil	200,000
Levorphanol	Levo-Dromoran	10,000
Levothyroxine (T ₄)	Synthroid	50,000
Meperidine	Demerol	800,000
Methadone	Dolophine	500,000
Methamphetamine	Desoxyn	500,000
Morphine		9,000
Morphine-3-Glucuronide		600,000
Morphine-6-Glucuronide		600,000
Nalorphine		7,000
Naloxone	Narcan	300,000
Naltrexone	Depade, ReVia	300,000
Nifedipine	Procardia	500,000
Norcodeine		600,000
Nordiazepam (Desmethyldiazepam)		100,000
Normorphine		30,000
Oxazepam	Serax	250,000
Oxycodone	OxyContin	400,000
Oxymorphone	Numorphan	80,000

Negative Compounds	Trade Name	Concentration Tested (ng/mL)
Perphenazine	Trilafon	150,000
Petazocine HCL	Talwin	350,000
Phencyclidine (PCP)		30,000
Phenobarbital	Luminal	500,000
Procyclidine	Kemadrin	800,000
Propoxyphene		500,000
Protriptyline	Vivactil	200,000
Ranitidine	Zantac	500,000
Salicyluric Acid		500,000
Secobarbital	Seconal	500,000
Tripolidine	Actagen, Allerfrin, Aprodine	50,000
Verapamil	Isoptin, Verelan	500,000