

Cross-reactivity nal von minden Drug-Screen® Drug Rapid Tests Urine

The following substances might can cause a cross-reaction with the nal von minden Drug Screen® Rapid Drug Tests Urine, or has been tested to lead to no cross-reaction. The list includes the substances that have been tested until now and does not claim to be complete.

Explanations:

Column "Positive":

There is the possibility that the substance cause a Positive result with the parameter listed drug test after consumption.

Column "No cross-reaction":

- Parameters: The substances should not have any influence on the listed parameters.
- No cross-reaction: The substances should not have any influence on any of our parameters.

Is a parameter stated in "positive" and "no cross.-reaction" means that it cannot be excluded that the substance influence the tests and it cannot be excluded that does not have any influence on the test.

Substance	Medicament	Positive	No cross-rection
1,2-Dimethoxybenzene			No cross reaction
Abacavir	Kivexa, Trizivir, Ziagen	MOR/OPI	
AB-FUNACA		Spice/K2	
Acramposat	Aotal, Campral		No cross reaction
Paracetamol / Acetyl-salicylic acid / Coffeine	Chephapyrin, Citramon, dolomo, Dolopyrin, Excedrin, HA-Tabletten N, InfluASS, Irocophan, Melabon, Neuralgin, Novo Petrin, ratiopyrin, Thomapyrin, Titalgan	PCM	
Paracetamol / Codeine	Aceta w/Codein, Co-codamol, Codafen, Co-dydramol, Kapake, Tylenol + Codeine, Tylenol 2, Tylenol 3, Tylex	PCM, MOR/OPI	
Acetaminophene / Coffeine	Excedrin Quick Tab Peppermint, Excedrin Quick Tab Spearmint, Excedrin Tension Headache, Excedrin Tension Headache Caplet, Excedrin Tension Headache Express Gels, Excedrin Tension Headache Geltaf, Pyrenol, Valorin Extra	PCM	
Paracetamol / Dihydrocodeine tartrat	Paramol	PCM, MOR/OPI, BUP	

Schnelltests

Labordiagnostika

Laborservice

Beratung & Service

nal von minden GmbH

Carl-Zeiss-Straße 12
47445 Moers
Deutschland

Tel.: +49 941 29010-0
Fax: +49 941 29010-50

Friedenstraße 32
93053 Regensburg
Deutschland

info@nal-vonminden.com
www.nal-vonminden.com

Geschäftsführer:
Sandra von Minden
Roland Meißner
Thomas Zander

Registergericht Kleve:
HRB 5679
Steuer-Nr. 244/133/00130
UST-ID-Nr. DE 189 016 086

Sparkasse Re-
gensburg
Kto: 840 017 073
BLZ: 750 500 00

Deutsche Bank
Kto: 056 998 800
BLZ: 350 700 24

Cross-reactivity nal von minden Drug-Screen® Rapid Tests Urine

Substance	Medicament	Positive	No cross-rection
Acetamiophene / Diphendydramine	Tylenol PM	PCM, MTD, EDDP	
Acetaminophene / Paracetamol / Methionine	Paradote,	PCM	
Acetaminophene = Paracetamol	Acamol, Ace +, Acephen, Acetalgin, Adol, Aldolor, Alvedon, Apacet, APAP, Apiretal, Apiretal Flas, Atamel, Atasol, Benuron, ben-u-ron, Biogesic, Biogesic-Kiddielets, Buscapina, Calpol, Captin, Cemol, CoEfferalgan, Coldex, Contac, Contra-Schmerz, Cotibin, Crocin, Dafalgan, Daleron, Dapacem, Dawa ya magi, Depon, Dexamol, Dexamol Plus, Dolex, Dolgesic, Doliprane, Dolprone, Duiyixian anjifen pian, Efferalgan, Enelfa, Enelfa Dr. Henk, Europain, Excedrin (combination), Febrectal, Febricet, Febridol, Fensum, Feverall, Fortagesic, Gelocatil, Gesic, Gripin, Grippex, Hedanol, Hedex, Herron, Influbene, Kafa, Kitadol, Lekadol, Lupocet, Metacin, Mexalen, Milidon, Minoset, Momentum, Napa, Neo-Kiddielets, Pacimol, Pacol, Pain Relief, Pamol, Panadol, Panamax, Panda, Panodil, Para, Paracet, Paracitol, Paralen, Pamed, Parol, Perdolan, Perfalgan, Perfalgan, Pinex, Plicet, PyongSu Cetamol, Pyrigesic, Reliv, Revanin, Rokamol, RubieMol, Rubophen, Sara, Scanol, Sinpro N, Tachipirin, Tachipirina, Tafirol, Tapsin, Temptra, Termalgin, Tipol, Togal Classic Duo, Treuphadol, Triaminic, Tylenol, Uphamol, Vermidon, Vitamol, Xumadol, Zolben	PCM	
Acetophenetidine	Phenacetin		No cross reaction
Acetyl salicylic acid, ASS	Anadin, Angettes, ASA, Asasantin, Aspirin, Bayer Aspirin, Bufferin, Caprin, Caprin, Disprin, Ecotrin, Empirin, Excedrin, Excedrin (combination)		No cross reaction
Afentanil / Natriumchloride	Alfenta, Rapifen		No cross reaction
Alimemazine = Trimeprazine	Nedeltran, Nedeltran, Panectyl, Repeltin, Temari, Therafene, Theralen, Theralene, Theraligene, Vallerган, Vanectyl	TCA	MPD
Allobarbital	Cibalen, Cibalgин, Curral	BAR	
Alprazolam	Tafil, Xanax, Xanax retard, Xanor	BZD	
Aluminiumhydroxide	Anhydrol Forte, Drichlor, Alisone, Alu-Cap, Gastrocote, Kolanticon, Maalox, Maalox TC, Mucogel, Pyrogastrone, Topal		No cross reaction
Aluminumhydroxide / Belladonna / Kaolin Light / Morphinehydrochloride	Opazimes	MOR/OPI	
Alverinzitrat	Spasmonal, Spasmonal Forte		No cross reaction
Amisulpride	Amazeo, Amival, Solian, Soltus, Sulpitac	BUP	
Amitriptyline	Amineurin, Elavil, Lentizole, Saroten, Triptafen, Triptafen-M, Tryptizol	TCA	
Amitriptyline / Chlordiazepoxide	Limbitrol	TCA	
Ammonia / Brechwurzel	Ipecac		No cross reaction

Kreuzreaktivitätsliste nal von minden Drug-Screen® Schnelltests

Substance	Medicament	Positive	No cross-rection
Amobarbital	Dorlotyn, Sodium Amytal, Tuinal	BAR	
Amoxicilline	Amoram, Amoxil, Augmentin, Heliclear		No cross reaction
Ampicilline	Penbritin, Polycillin, Principen		No cross reaction
Antazolinsulphat	Otrivine-Anistin		No cross reaction
Aprobarbital	Alurate	BAR	
Aripiprazol	Abilify	PCM	AMP, MET, MDMA, MDA, MTD, THC, COT, MQL, BUP, BAR
Articaine HCl	Ultracain		MET, AMP
Atenolol	Atedurex, Atehexal, Atenil, Atenolan, Atenolol, Beta-adlat, Bresben, Cardaxen, Cardaxen plus, Co-tenidone, Cotenolol, Diu-Atenolol, Juvental, Kalten, Nifatenol, Nif-Ten, Selobloc, Tenben, Teneretic, Tenorectic, Tenormin, Tenoret50		No cross reaction
Atomoxetine Hydrochloride	Strattera	MPD	
Atorvastatine	Atorvalan, Lipitor, Sortis, Tahor	EDDP	
Azocilline	Securoopen		No cross reaction
Baclofene			BZD
Barbital	Medinal, Veronal	BAR	
Beclometasone	AeroBec, AeroBec Forte, Asmabec, Beclazone, Becloforte, Beclomet, Becodisks, Beconase, Becotide, Fillair, Nasobec, Otriven Allergie Aktiv BEC, Qvar, Ventide, Zonivent		No cross reaction
Bendroflumethiazide	Aprinox, Cogaretic, Inderex, Indertic, Neo-Naclax, Neo-Naclax-K, Prestim, Tenben		No cross reaction
Benperidol (Butyrophenon)	Glianimon		No cross reaction
Benzalkonium Chloride	Bradosol, Sagrotan		No cross reaction
Benzydamin HCl	Difflam, Crystapen		No cross reaction
Bezafibrat	Cedur®	EDDP	AMP, MET, MDMA, MDA, KET, BZD, TML, COC, MOR/OPI, MTD, THC, COT, TCA, OXY, FYL, PPX, BUP, BAR
Biperiden	Akineton		No cross reaction
Bisoprolol	Bilol, Bisacardiol, Bisocor, Concor, Concor COR, Jutabis, Rivacor		TCA
Bromazepam	Lexotan	BZD	
Bromopheniramine	Dimetapp, Dimotane, Dimotapp, Budapp		No cross reaction
Budesonide			MDMA/XTC, BZD
Buprenorphine	Mylan 8 mg, Suboxone, Subutex, Temgesic	BUP	
Bupropione	Elontril 300, Wellbutrin, Zyban		No cross reaction
Busprione	Anxut, Busp		MPD
Butabarbital	Butethal, Butisol, Soneryl	BAR	
Butalbital	Fioricet, Fiorinal	BAR	
Butylscopolamin Bromid	Buscopan, Spasman		BZD, MOR/OPI, BZD
Calciumchloride	Artisial		No cross reaction
Carbamazepine	Tegretol, Finlepsin, Tegretal, Teril, Timonil		No cross reaction

Cross-reactivity nal von minden Drug-Screen® Rapid Tests Urine

Substance	Medicament	Positive	No cross-rection
Carfentanyl	Wildnil		No cross reaction
Carvedilol	Dilatrend	MOR/OPI	MOR/OPI
Cefalexine	Cephalex, Ceporex, Keflex	MOR/OPI	
Cerufloxim	Elobact	COC	COC
Cetirizin	Cetirizin Ratiopharm, Zyrtec	BZD, TCA	
Chloralhydrate	Chloraldurat 500, Welldorm		No cross reaction
Chlorazepat	Tranxene	BZD	
Chlordiazepoxide	Librium	BZD	
Chlorhexidine	Antebor, Bepanthen Antiseptische Wundcreme, Bepanthen Plus, Cervitec liquid, Cervitec Plus, Cervitec Gel, Collu-Blache, Collunosol, Eludril, Hibital alkoholische Lösung/ -Tinktur, Instillagel, Lindosan Wund- und Heilsalbe, Luuf Halspastillen/ -Rachenspray, Merfen, Neo-Angin Spray, Nystalocal, Skinsept F/ -mucosa, Vitaderm, Vita-Hexin, Vita-Merfen, Vitawund, anti-infect Dental-spray, Curasept ADS, Paroguard, Paroex, Chlorhexamed, Cidegol, Corsodyl, Dentohexin, DermaPlast, Dynexan Proaktiv, Hexal-Lösung, Hibidil Sterile Lösung, Hibiscrub, Hibitane Konzentrat, Lifo-Scrub, Meridol, Uro-Tainer, ChloSite Gel, PerioChip, Bactrigras, Chlorohex, Cordsodyl, CX Powder, Dermol, Hibicet, Hibiscrub, Hibisol, Hibitane, Instillagel, Naseptin, Nystaform, Serotulle, Steripod, Tisept, Unisept, Uriflex, Uro-Trainer		
Chlorpheniraminmaleat	Balkis Schnupfenkapseln Neu, Galpseud, Haymine, Piriton		No cross reaction
Chlorpromazine	Largactil		No cross reaction
Chlorprothixene	Truxal	TCA	
Cimetidine	Dyspamet, Tagamet, Zita		No cross reaction
Cinoxacine	Cinoxacin	MOR/OPI	
Citalopram	Cipramil, Cipram, Seropram	BZD	
Clarithromycine	Clarithrocin-Mepha		No cross reaction
Clobazam	Frisium	BZD	
Clonazepam	Clonopin, Klonopin, Rivotril	BZD100	
Clotrimazol	Antifungol, Canesten ears/skin/vaginitis/HC		No cross reaction
Clozapin	Clopin, Leponex	TCA	BZD
Co-Amoxiclav	Augmentin, Augmentin Duo		No cross reaction
Codein + Paracetamol	Pancos	PCM, OXY	
Codein Phosphat	Codafen Continus, Codeine Linctus, Galcodine, Kapake, Migraleve, Pediatric BP, Solpadol, Tylex	MOR/OPI, BUP	
Co-Phenotrop (atropin/diphenoxylat)	Lomotil, Trepergen		No cross reaction
Cromoglicinsäure			MDMA/XTC
Cyamemazin	Tercian	TCA, PPX, PCP, COC, MOR/OPI	
Octapamin	Norsynephrine		No cross reaction
Darunavir		COC	
Delorazepam	Briantum	BZD	
Dexamethason	Decadron, Dexa-Rhinaspray Duo, Maxidex, Maxidrol, Minims, Otomize, Sofradex		No cross reaction
Dexamphetamin Sulphate	Adderall, Adderall XR, Dexedrine	AMP	

Kreuzreaktivitätsliste nal von minden Drug-Screen® Schnelltests

Substance	Medicament	Positive	No cross-rection
Dexchlorpheniramine / Paracetamol	a-ferin	MTD	MOR/OPI
Dexketopropofene			THC
Dextromethorphan	NeoTussan, Silomat DMP, Tussastopp, Wick Hustenpastillen/ Hustensirup, Bexin, Calmerphan		No cross reaction
Dextromoramide	Palfium		OPI/MOR
Dextropropoxyphene	Darvocet, Darvon, Co-proximal	PPX	
Diazepam	Diazemuls, Stesolid, Valclair, Valium	BZD	
Diclofenac Sodium	Dicloflex, Diclomag, Motifene, Volraman, Volsaid, Voltarol		No cross reaction
Dicycloverin	Kolanticon, Merbentyl		No cross reaction
Dihydrocodeine	Codicontin, Codidol, Codidol ret., Dehace, DHC, DHC Continus, Escotussin, Makatussin comp., Paracodin, Tiamon	BUP, MOR/OPI	
Paracetamol / Dihydrocodeine tartrat	Paramol, Remedeine	BUP, MOR/OPI, PCM	
Dimenhydrinat (Diphenhydramine+Theophyline)		MTD, EDDP	
Dimethicone	Asilone, Ceolat SF, Fancorsil		No cross reaction
Diphenhydramie	Beatdorm, Benadryl, Medinex, Nytol, Panadol Night, Vivirnox sleep	MTD, EDDP	
d-Methamphetamine HCL	Desoxyn, Methamprex, Methedrine	MET	
Domperidone	Motilium, Motillium, Motinorm Costi, Nomit		MOR/OPI
Dosulepine	Idom, Prothiaden		No cross reaction
Doxepin	Aponal, Doxepin, Sinequan, Xepin	TCA, MOR/OPI	TML, MDMA, THC
Doxycycline	Vibramycin, Vibramycin-D		No cross reaction
Doxylamine	Gittyl, Wick MediNait		BZD
Doxylamine Succinat	Hoggar Night	MTD, COC, TCA, PCP, PPX, BUP	BZD
Dronabinol (Delta-9-Tetrahydrocannabinol)	Marinol	THC	
Droperidol	Dropletan, Xomolix		No cross reaction
Ebastine	Ebastel		MET
Efavirenz	Stocrin, Sustiva	THC	
Emtricitabine			COC
Ephedrine	Caniphedrin, Helopyrin	MET	AMP
Erythromycine	Arpimyclin, Erycane, Erymax, Erythrocin, Erythroped, Ilosone, Isotrexin, Lactobinate, Stiemycin, Tiloryth, Zineryl, Benzymycin		No cross reaction
Escitalopram	Cipralax	BZD, MPD	BUP
Esomeprazol	Durotiv, Esomep, Inexium, Nexium	BZD	TCA
Estazolam	ProSom	BZD	
Ethambutol	Myambutol		No cross reaction
Ethylmorphin	Dionin	MOR/OPI	
Fenofibrat	Procetofen		No cross reaction
Fentanyl	Actiq	FYL	
Fexofenadin	Telfast	TCA, TML	
Fleroxacin	Quniodis	MOR/OPI	
Flucloxacillin	Co-fluampicil, Floxapen, Magnapen		No cross reaction

Cross-reactivity nal von minden Drug-Screen® Rapid Tests Urine

Substance	Medicament	Positive	No cross-rection
Flumazenil	Anexate, Lanexat, Mazicon, Romazicon	BZD	
Flunitrazepam	Rohypnol	BZD	
Fluoxetine	Fluoxetin, Prozac, Sarafem	BZD	
Flupentixol decanoat	Depixol, Fluanxol		No cross reaction
Flupirtine	Katadolon		AMP, MET
Folic acid	Folsan		AMP, BAR, BZD, BUP, THC, COT, MDMA, EDDP, FYL, KET, COC, MDA, MTD, MET, MQL, MOR/OPI, OXY, PCM, PCP, PPX, TML, TCA
Formoterol	Foradil, Forair, Formatrix, Oxis		COC, EDDP, MOR/OPI, BZD, MDMA/XTC
Fusidin acid	Fucibet, Fucidin, Fucidine, Fucithalmic, Fusicutan		No cross reaction
Gabapentin	Gabatal, Gababurg, Gabagamma, GabaLich, Gabalster, Gabantin, Gabax, Neurotin	MOR/OPI, MPD	BZD
Gentamicinsulphate	Septopal, Cidomycin, Garamycin, Gencin, Gentamycin, Gentamytrex, Gentax, Genticin, Gentisone, Gent-Ophtal, Ophtagram, Refobacin, Sulmycin, Terramycin		No cross reaction
Haloperidol	Haldol, Serenase		No cross reaction
Hydrocodone	Loratab, Lorcet, Vicodin	MOR/OPI	MOR/OPI
Hydrocortisone	Actinac, Alphaderm, Alphosyl-HC, Anugesic-HC, Anusol-HC, Calmurid-HC, Canesten-HC, Colioam, Daktacort, Dioderm, Econacort, Efcortesel, Efortelan, Eurax-HC, Gregoderm, Hydrocortistab, Hydrocortone, Mildison lipocream, Neo-cortef, Nystaform, Perinal, Proctofoam, Quinocort, Terra-cortil, Timodine, Uniroid-HC, Vioform-HC, Xyloproct		No cross reaction
Hydrocortison-17-butytrat	Locoid, Locoid C		No cross reaction
Hydromorphone	Dilaudid, Hydal, Hydrostat IR, Jurnista, Palladon	MOR/OPI	
Hydroxychloroquine	Plaquenil, Quensyl		No cross reaction
Hydroxyzine	Atarax, Ucerax, Vistaril	THC, MTD, MPD	
Ibuprofen	Brufen, Codafen, Dolormin, Fenbid, Ibugel, Ibuspray, Motrin, Proflex, Spalt		No cross reaction
I-Methamphetamine HCL	Vicks Inhaler	MET	
Indomethacine	Elmetacin, Flexin continus, Indo Top, Indocid, Indoclor, Indocollyre, Indomet Schmerzspray, Indomod, Indo-paed, Indophtal, Luiflex, Rheubalmin Indo Spray, Vonum		No cross reaction
Interferon Alfa	Intron A, Multiferon, Roferon-A, Viraferon, Wellferon		No cross reaction
Isoniazide	INH, Isozid, Rimifon, Tebesium		No cross reaction
Isosorbiddinitrat	Angitak, Carvasin, Cedocard, Diconpin, Isocard, Isoket, Isomack, Iso-Puren, Isordil, Jenacard, Nitrosorbon, Sorbichew, Sorbid SA, Sorbidilat, Sorbitrate		No cross reaction
Isotretionine	Aknenormin	BZD	

Kreuzreaktivitätsliste nal von minden Drug-Screen® Schnelltests

Substance	Medicament	Positive	No cross-rection
Kaolin and Morphine Mixture	Entersan	MOR/OPI	
Ketoprofene	Alrheumon, Dolormin Schmerzgel mit Ketoprofen, Effekton Gel mit Ketoprofen, Fastum Gel, Gabrilen, Ketospray, Orudis, Oruvali, Phardol, Powergel, Profenid, Prontoket, Spondylon		No cross reaction
Khat		AMP	AMP
Lactulose	Bifinorm, Duphalac, Lactugal		
Lamivudine	Epivir, Kivexa, Trizivir	MOR/OPI	
Lamotrigine	Lamictal		No cross reaction
Lansoprazol	Agopton, Heliclear, Lansazol, Zoton		No cross reaction
Levetiracetam	Keppra		No cross reaction
Levofloxacin	Tavanic	MOR/OPI	
Levomepromazine	Levium, Neurocil, Nozinan	COC, TCA	MOR/OPI
Levomethadone	L-Polamidon	MTD	
Levothyroxine	Euthyrox, L-Thyrox Hexal		AMP, MET, MDMA, MDA, KET, BZD, TML, COC, MOR/OPI, MTD, EDDP, THC, COT, TCA, OXY, FYL, MQL, PCP, PPX, BUP, BAR, BZD
Thyroxine	Eltroxin, Euthyrox, Eutirox, Letrox, Levaxin, Levoxyl, L-thyroxine, Synthroid, Thyrax, Thyrox Duotab, Thyrin, Thyrolar, Thyrox, Tirosint		No cross reaction
Levothyroxine sodium	Eltroxin		No cross reaction
Lidocaine	Versatis	BZD	BZD
Lisdexamphetamine	Elvanse, Tyvense, Venvanse, Vyvanse		No cross reaction
Lisinopril HCl	Acerbon, Carace, Zestoretic, Zestril		No cross reaction
Lithium	Téralithe		No cross reaction
Lofepamine	Gamonil, Lomont	TCA	TCA
Lofexidine	BritLofex		No cross reaction
Loperamid HCl	Imodium, Loperagen, Norimide		No cross reaction
Lopinavir	Kaletra		COC, BZD, MTD, EDDP, MOR/OPI
Loratadine	Claritin, Loratadin		No cross reaction
Lorazepam	Ativan	BZD100	
Lormetazepam	Noctamide, Noctamide	BZD100	
L-Thyroxin	Euthyrox		FYL
Lysergic acid diethylamide	LSD		No cross reaction
Magnesium Alginat	Gaviscon		No cross reaction
Magnesium Chlorid Hexahydrate	Artisial		No cross reaction
Mannitol	Gaviscon, Osmofundin, Osmosteril		No cross reaction
mCPP			TCA
Medazepam	Anxitol, Lerisum, Medacepan, Nobritol, Nobrium	BZD	
Mefenamin acid	Ponstan, Ponstel	THC	
Melatonine			MPD, BZD, THC
Eunerpane			

Cross-reactivity nal von minden Drug-Screen® Rapid Tests Urine

Substance	Medicament	Positive	No cross-rection
Meperidine	Demerol, Dolantin	DOL	
Mepivacaine	Meaverin, Mecain, Mepinaest, Scandicain, Scandonest 3 % ohne Vasokonstriktor	MOR/OPI	
Metadoxine	Abrixone, Alcotel, Ganxin, Metadoxil, Viboliv, Xin Li De	THC	
Metamizole	Metapytine, Novaminsulfon		BZD, COC
Metformine	Glucophage, Metformin		No cross reaction
Methadone	Polamidon	MTD	TML, MOR/OPI, BZD
Methadon-Hydrochloride	Dolophine, Methadose, Physetone, Polamidon	MTD	
Methorcarbamol	Methorcarbamol 750		BZD
Methotrexate	Bendatrexat	BAR, BUP, THC, COT, FYL, KET, MQL, MOR/OPI, OXY, PCM, PCP, PPX, K2, TML, TCA	AMP, BZD, BUP, THC, COT, MDMA, EDDP, EtG, FYL, KET, COC, Crea, MDA, MTD, MET, MQL, MPD, MOR/OPI, OXY, PCP, K2, TML, TCA, ZOL
Methoxyisoflavone	Methoxyisoflavone		THC
Methylendioxyamphetamine (MDA)	Eve (slang), Love Drug (slang), XTC (slang)	AMP, MDMA	
Methylendioxymethamphetamine (MDMA)	Adam (slang), E (slang), Ecstasy	MET, MDMA	
Methylphenidate	Medikinet, Ritalin	MPD	
Metoclopramide	Reglan		No cross reaction
Metronidazol	Anabact, Elyzol, Flagyl, Metrogel, Metrolyl, Metrotop, Neuratop, Noritate, Rozex, Zidoval, Zymoet		No cross reaction
Milnacipram			No cross reaction
Mirtazapine	Remergil, Remeron	MOR/OPI, TCA	TML, THC, MPD, BZD
Moclobemid	Moclobemid	MOR/OPI	
Morniflumate			
Morphine	Astramorph, Cyclimorph, Duramorph, Morcap, Morphine Sulphate, MS Contin, Oramorph, Roxanol, Severedol	MOR/OPI	
Morphinhydrochlorid	Compensan ret., Vendal ret.	MOR/OPI	
Morphin hydrochlorid / Attapulgit / activated Attapulgit	Diocalm	MOR/OPI	
Morphinsulfat	Kapanol ret., Substitol ret.	MOR/OPI	
Moxifloxacin	Avalox	MOR/OPI	
Nadolol	Corgard, Corgaretic, Corgartic		No cross reaction
Naloxon Hydrochloride (2H2O)	Targin		MOR/OPI
Naloxone	Nalone, Narcan		MOR/OPI
Naltrexone	Antaxon, Antaxone, Nalorex, Trexan, Nalorex		No cross reaction
Naproxene	Aleve, Condrotec, Napratec, Naprosen, Naprosyn, Nycopren, Synflex		No cross reaction
Niastine			No cross reaction
Nicotinamide	Nicobion		No cross reaction

Kreuzreaktivitätsliste nal von minden Drug-Screen® Schnelltests

Substance	Medicament	Positive	No cross-rection
Nicotine	Nicoderm, Nicopatch, Nicorette, Nicotinell, Nicotrol, Niquitin	COT	
Nifedipine	Adalat, Nifar, Nifecard, Nif-ten, Procardia		No cross reaction
Niflurique acid	Nifluril	THC	
Nitrazepam	Mogadon, Somnite	BZD	
Nitrite			No cross reaction
Norethindrone	BiNovum, Brevinor, Climagest, Climesse, Elleste Duet, Estracombi, Evorel, Kiofem, Kioavance, Loestrin, Micronor, Noriday, Norimin, Norinyl, Noristat, Norlutin, Nuvelle, Ovysmen, TriNovum, Trisquens, Utovian		No cross reaction
Noscapine	Narcotine		No cross reaction
Novaminsulfone	Novaminsulfon		BZD
Ofloxacin	Floxal, Tarivid	MOR/OPI	
Olanzapine	Zyprexa		No cross reaction
Opamox			No cross reaction
Opipramol	Isidon	TCA	
Orphenadrine	Disipil, Norflex, Norgesic		No cross reaction
Östrogen/Gestagen	Valette		BZD
Oxazepam	Ox-pam, Serax	BZD	
Oxcarbazepine	Apydan Extent, Timox, Trileptal	TCA	THC
Oxycodone	Oxycotin, Oxygesic	MOR/OPI, OXY	
Oxycodone Hydrochloride	Targin	OXY	
Oxymetazoline	Afrin, Neo-syneprine, Terra-Cortil, Terramycin, Trimovate		No cross reaction
Pantoprazol	Gastrozol, Pantozol, Zurcal		No cross reaction
Pantozol	Pantozol Control		No cross reaction
Paracetamol	Doliprane	PCM	
Paracetamol + Codeine Phosphat	Co-Efferalgan	PCM, BUP, MOR/OPI	
Paroxetine	Paxil, Seroxat		No cross reaction
Penicilline	Combicillin, Mefoxin		No cross reaction
Pentobarbital	Nembutal	BAR	
Perazin-Dimalonat	Perazin-neuraxpharm, Taxilan		No cross reaction
Pericyazine	Amplan, Aolept, Apamin, Iryakin, Nemactil, Neulactil, Propetyl, Psycholept		No cross reaction
Pervitine	Pervitin	MET	
Pethidine (Meperidine)	Demerol, Dolantin	DOL	
Phenazepam	Phenazepam	BZD	
Phenobarbital = Phenobarbitone	Donnatal, Luminal	BAR	
Phentermine	Phentermin	AMP, MET	
Phenylephrine			BZD
Phenylpropanolamine	Wick Daymed	AMP	
Phenytoine	Dilantin, Epanutin, Epitard, Phenhydan, Zentropil		No cross reaction
Phloroglucinol	Spasfon		No cross reaction
Pholcodine	Galenphol, Pavacol-D, Strong BP, Thebacon	MOR/OPI	
Piperacilline	Pipracil, Tazobactam		No cross reaction
Piritramide	Dipidolor	EDDP, MOR/OPI	

Cross-reactivity nal von minden Drug-Screen® Rapid Tests Urine

Substance	Medicament	Positive	No cross-rection
Kaliumchlorid / Kaliumphosphat Dibasisch / Kaliumphosphat Monobasisch	Artisial		No cross reaction
Prazepam	Centrax, Demetrin	BZD	
Prednisolone	Delta-cortef, Econopred, Inflamase, PEDIAPRED, Prelone		No cross reaction
Pregabalin	Lyrica		No cross reaction
Procaine	Novocain	MOR/OPI	
Progesterone	Utrogest		No cross reaction
Promethazine	Allerfen, Anergan, Antinaus, Aprobit, Atosil, Avomine, Baymethazine, Diprozin, Fenegan, Methazine, Phenergen, Propavan	TCA	
Propofol			No cross reaction
Propral			No cross reaction
Propranolol HCl			TCA
Propranolol	Beta-Progane, Inderal, Inderetic, Inderex	MPD	THC
Prothipendyle	Dominal	TCA, MOR/OPI	
Pseudoephedrine	Afrinol, Aspirin Complex Granulat, Boxagrippal, Rhinopront Kombitabletten, Sudafed, Tylenol Cold (combination)	MPD	MET300, AMP300, MDMA500
Pyridostigmin Bromid	Kalymin		BZD
Pyridoxin (Vitamin B6)	Becilan		No cross reaction
Quetiapine	Seroquel	TCA	
Rabeprazol	Pariet	BZD	No cross reaction
Raltegravir	Isentress		THC
Ranitidine	Pylorid, Zantac	MET	
Reboxetine	Edronax, Solvex		MET
Remifentanyl	Ultiva		No cross reaction
Reproterol			MDMA/XTC
Resorcinol			No cross reaction
Rifampicine		MOR/OPI	MOR/OPI
Risperidone	Risperdal Consta, Risperidal	BZD	
Ritanovir	Norvir, Kaletra	THC	COC, BZD, MTD, EDDP, MOR/OPI
Salbutamol	Aerocrom, Aerolin, Airomir, Asmasal, Buventol, Combivent, Duovent, Venotlin, Ventide, Ventodisks, Volmax		COC, EDDP, MOR/OPI, BZD
Saliva			THC
Butylscopolaminbromide	Busocopan, Scopoderm, Spasman		No cross reaction
Secobarbital	Seconal	BAR	
Sennosides	Senokot		No cross reaction
Seropram	Citalopram	BZD	
Sertraline	Sertraline Arrow, Zoloft	BZD	
Natriumchloride	Artisial		No cross reaction
Natriumvalproat	Convulex, Depakene, Depakote, Epilim, Ergenyl, Orfiril		No cross reaction
Sucralfat	Sucrabest, Ulcogant		TCA
Sufentanil	Sufenta		No cross reaction
Sulfamethoxazol	Bactrim, Cotrim, Drylin, Escoprim, Eusaprim, Kepinol, Lagatrim, Nopil		

Kreuzreaktivitätsliste nal von minden Drug-Screen® Schnelltests

Substance	Medicament	Positive	No cross-rection
Tapentadol		MTD	BUP
Tapentadol Hydrochlorid	Palexia		No cross reaction
Temazepam	Restoril	BZD	
Tenofovir			COC
Testosterone	Andropatch, Restandol, Sustanon, Virormone		No cross reaction
Thiamine (Vitamin B1)	Bevitine		No cross reaction
Thioridazin Hydrochloride	Mellaril		No cross reaction
Thiorphan	Thiorphan		MOR/OPI
Tiapride	Tiapridal		No cross reaction
Tilidine	Tilidin, Valtran, Valoron		No cross reaction
Tilidine / Naloxone	Valoron N	MOR/OPI	
Topiramat	Topamax, Topiramat		No cross reaction
Tramadol	Adamon, Amadol, Contramal, Cromatodol, Dolevar, Ecodolor, Jutadol, Lanalget, Noax, Nobligan, T-long, Tradolan, Tradonal, Tramagit, Tramake, Tramal, Tramundin, Travex, Ultram, Zaldiar, Zamadol, Zydol	TML, PCP	MOR/OPI, BUP, OXY
Tramadol Hydrochlorid	Zydol	TML	
Trazodon	Beneficat, Deprax, Desirel, Desyrel, Mesyrel, Molipaxin, Oleptro, Thombran, Trazorel, Trialodine, Trittico	TZD	TCA
Triazolam	Halcion	BZD	
Trimethoprime	Bactrim, Cotribene, Cotrim, Drylin, Eusaprim, Infectotrimet, Kepinol, Lidaprim, Motrim, TMS forte, Triprim	BUP, BZD	
Trimethoprime / Sulfamethoxazole	Co-trimoxazol	BZD	
Trimipramine	Herphonal, Stangyl, Surmontil	TCA, MOR/OPI	BUP, MOR/OPI
Tropatepine	Lepticur	TCA, COC	
Ergenyl Chrono			THC
Valproic Acid	Convulex, Convulsofin, Depakine, Depakine Chronosphere, Ergenyl, Leptilan, Natriumvalproat, Orfiril, Valproat		No cross reaction
Valpromide	Dépamide		No cross reaction
Venlafaxine	Effexor, Effexor XL, Trevilor, Venlafaxin Abz, Venlafaxin Atid	PCP, MPD, BZD, TML	MTD
Warfarin Sodium	Coumadin, Marevan		No cross reaction
Xylometazolin	Olynth, Otriven		No cross reaction
Zaleplone	Andante, Sonata, Starnoc	ZAL	ZOL
Zidovudine	Trizivir		
Zolpidem	Stilnoct	ZOL	BZD
Zopiclone	Ximovan, Zop Filmtablette, Zopiclon		No cross reaction

Please take into consideration that only the mother substance in high concentrations and no metabolite, which could account for the majority of substance in the urine, was analyzed. Thus, the results must be seen as support to interpret the test but not necessarily represent the situation *in vivo*.